


Viewer Main Toolbar

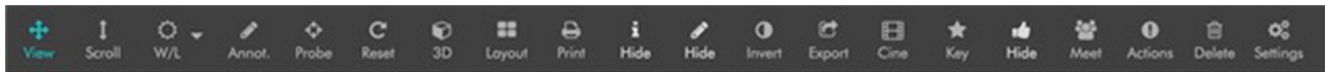

View

Toggles the View sub-toolbar visible/hidden. See sub-toolbar button descriptions on next page.

Scroll

Scrolls through a study's images. Use mouse wheel to scroll.

Window Level

Sets image's window width (contrast) and window level (the number at the center of the window). Mouse: Window width: hold left click, move mouse left-right. Window level: hold left click, scroll up-down. Touch pad: Window width: one finger tap, drag left-right. Window level: one finger tap, drag up-down. See window width/level description on a subsequent page.

Annotate

Toggles Annotation sub-toolbar visible/hidden. See annotation sub-toolbar button descriptions on a subsequent page.

Probe

This toolbar button appears only when viewing CTs. Click and drag on an image to read Hounsfield values.

Reset

Returns series to original state while remaining on current image. Resets window level, invert tone, rotation, zoom, free draw, and line measure annotations.

3D

Toggles 3D sub-toolbar visible/hidden.

Layout

Choose number of images to display from the grid. 'Maximize' opens the selected series in a one-up full screen view. 'New' opens the study in a new browser tab.

Print

Opens a page in a new browser window with a hyperlinked list of all available images in the study(s) being viewed. See a subsequent page for how-to-use instructions.

Info

Toggle to show/hide study information.

Annotations visible/hidden

Toggle to show/hide annotations.

Invert

Inverts image colors.

Export

Export png, create a secondary capture, view the study meta data in a new tab.

Cine

View images as a movie. US (multi-frame) studies are displayed with HTML5 video in actual frames per second. See Cine Sub-Toolbar in Clinical Viewer Guide.

Key

Makes the visible study image (high-lighted with a blue bounding box), a Key image. A study's key images can be viewed by selecting 'View Key Images' in the view options menu in the studies list.

Thumbs

Toggles study thumbnails visible/hidden.

Meet

Start a live collaborative meeting with other DG users.

Actions

'Share' shortcuts. See sub-toolbar description in Clinical Viewer Guide.

Delete

Delete images or series. See sub-toolbar description in Clinical Viewer Guide.

Enhance

Locally enhance the contrast of an image. This filter provides an alternative to the window/level filter for global contrast enhancement.